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The Solution for Scars™

Silicone Scar Sheets

Silicone sheet technology is clinically proven and used by Plastic Surgeons, Hospitals and Burn Centers around the world.

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LATEX & DRUG FREE
8 Sheets - 1.5" x 3"

For use in the treatment and prevention of raised and discolored scars

Patented
Silon®
Technology

Thank you for selecting ScarAway®, the only product available in stores with patented Silon® technology and the latest innovation used by plastic surgeons, hospitals and burn centers. ScarAway shrinks, flattens and fades new and old scars, restoring skin to a more natural texture and color. Even scars that are years old show significant improvement. ScarAway is so advanced, it actually helps prevent unsightly scars from forming on newly healed skin.

Application and Care

1. Wash and dry scar area. Do NOT use a soap that contains moisturizers, lotions or oils. A plain basic soap is recommended.
2. Remove ScarAway sheet from its package.
3. If necessary, trim sheet to size for smaller scars prior to removing from liner. Wear side-by-side for larger scars. (ScarAway should extend beyond the scar area on all sides by at least ¼".)
4. Peel off liner from adhesive side of sheet.

5. Apply sheet with adhesive side directly on the scar. Press ScarAway sheet firmly onto scar area beginning at the center of the sheet and working toward each side.
6. Keep sheet in place for a minimum of 12 hours per day.
7. At least 1 time per day, remove ScarAway sheet. Wash the scar area and both sides of the sheet with a mild soap and water. Gently rub the silicone side with thumbs, working in a circular motion out toward the edges. Remember, use a moisturizer-free soap. Rinse ScarAway sheet thoroughly to remove all traces of soap. Allow sheet to air dry or pat dry with a lint-free cloth before reapplying. You can also place in the storage case* to dry with adhesive side down.

NOTE: Washing individual ScarAway sheets between wear actually helps to maintain the adhesive quality without affecting the silicone. Do not use any lotions, creams or oils under the ScarAway sheet as they may interfere with the performance of the product and/or cause skin irritation.

8. Store your ScarAway sheet on the protective liner or in your ScarAway storage case.*
9. After 7 days, discard ScarAway sheet and apply a new one following the above instructions.

*storage case available with select packs

NOTE: Medical tape or a cohesive wrap may be used to aid in securing the ScarAway sheet to your skin. This is recommended for joint and active areas or while sleeping. If your ScarAway sheet requires an additional means of adhesion to stay securely in place, this does not mean the sheet is any less effective. The strength of adhesiveness does not impact the effectiveness of the silicone.

Commonly Asked Questions

How does ScarAway work to prevent or minimize scarring?

ScarAway uses patented Silon® technology, a specially formulated silicone which mimics the natural barrier function of healthy skin. Research suggests that Silon acts to hydrate scar tissue, which in turn works to soften the scar, reducing its development and causing it to fade away. ScarAway also protects your delicate scar area and helps alleviate the itching, burning and pain that often accompany scars.

When should I begin using ScarAway?

For newly healed wounds, you may begin using ScarAway as soon as the skin is healed and there is no oozing or crusting in the area. Treatment with ScarAway for post surgical scars may typically begin approximately 1-2 weeks after stitches are removed or when cleared by your physician or surgeon for use.

For existing scars, you may begin using ScarAway right away, provided your skin is fully healed.

What types of scars does ScarAway work on?

ScarAway works on hypertrophic scars and keloids, which have a raised and/or discolored appearance.

Can I use ScarAway on my old (existing) scar?

Yes. ScarAway works on both old and new scars to shrink, flatten and fade your scar so it appears closer to your skin's natural color. ScarAway is even effective on scars that are years old.

Can I use ScarAway on any part of my body?

Yes. When used as directed, ScarAway silicone scar sheets can be used on any part of the body. In more active areas or around joints, medical tape or a cohesive wrap may be required to aid in adhering to the skin. This will not impact the silicone's effectiveness.

How long should I wear each sheet?

Each ScarAway sheet should be worn a minimum of 12 hours per day and up to 24 hours per day provided it is removed and the scar area and sheet are washed at least 1 time daily. The longer the silicone is in contact with your skin, the quicker you will see results. Each sheet is washable and durable for 7 days of use.

Can ScarAway be used on scars of different sizes and shapes?

Yes. ScarAway can be used on many different sizes and shapes of scars. ScarAway comes in 2 convenient sizes (1.5" x 3" and 1.5" x 7"). ScarAway sheets may be trimmed to size for smaller scars or worn side-by-side for larger scars. ScarAway should extend beyond the scar area on all sides by at least ¼".

How long will it take before I notice improvement?

ScarAway is a professional scar treatment that remodels your skin over time to reduce your scar and make it less noticeable. A softening of the scar may occur in just a few days. Improvement in scar color and thickness may be seen in as little as 4 weeks and will improve gradually over the treatment period. Best results can be achieved by preventing excessive scar formation in the first place. By using ScarAway immediately and consistently after your wound has healed, you will maximize your opportunity for optimal results.

How long do I need to use ScarAway?

Treatment duration will vary from person to person and from scar to scar, depending on many factors. Average treatment time for maximum results is 8-12 weeks. Further benefits may result from extended and consistent use. Individual results will vary.

Can I wear ScarAway while exercising or bathing?

It is not ideal as perspiration and many soaps, shampoos, etc. interfere with adhesion. It is recommended that sheets be removed and reapplied following these activities.

For answers to additional commonly asked questions and to learn more about ScarAway and facts about scars, please visit us at www.MyScarAway.com.

Cautions:

- This product is not sterile
- Keep out of reach of children under age 3 and pets, as sheets may present a choking hazard
- For external use only

As with any topical material applied directly to the skin's surface, some type of reaction (usually a sweat rash) is possible. In the unlikely event you have a reaction, stop using ScarAway. After your skin has returned to normal, you may reapply ScarAway for short periods and gradually increase the hours of usage. Monitor the scar area for signs of reaction. Continue to increase the wearing time until a comfortable maximum has been reached. If you continue to have a problem, discontinue use and consult your physician.

WARNING: DO NOT USE ON OPEN WOUNDS

Questions or Comments: Call 866-943-7225 Mon-Fri, 9am-5pm EST or visit www.MyScarAway.com.

Distributed by: Mitchell-Vance Laboratories LLC, Guilford, CT 06437
Licensed under patent numbers 5,656,279; 5,980,923 and 7,087,135.
Silon® is a registered trademark of Bio Med Sciences, Inc.
©2009 Mitchell-Vance Laboratories LLC || Made in the USA

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Distribuido por: Mitchell-Vance Laboratories LLC, Guilford, CT 06437
Silon® es una marca registrada de Bio Med Sciences, Inc.
©2009 Mitchell-Vance Laboratories LLC || Hecho en EE.UU